


Which Size Boot Do I Need?

Get the right size the first time by following these simple steps


- 1** Find a sturdy piece of paper or cardboard and place your horse's hoof squarely on it.

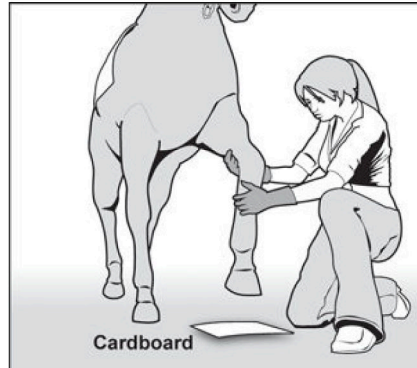

- 2** Use a short dark pencil (a golf pencil, for example). It must be able to run under the ankle at the back.


Trace around the foot holding the pencil as shown. You will need a tight trace with the pencil vertical. Trace around the heel bulbs.

Mark end of ground contact area on both sides of the hoof. (Shod or unshod) Ground contact area must fit within gel orthotic surface area.

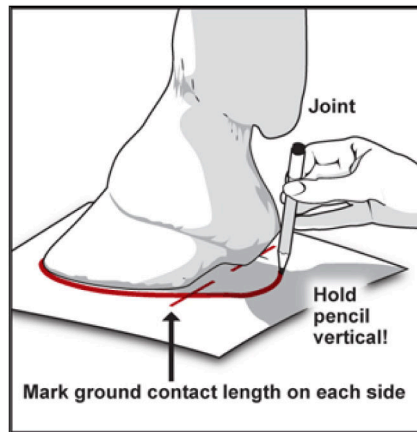

- 3** Now measure your trace at:

- The widest point, as shown
- The longest ground contact point, as shown.
- The longest point including bulbs, as shown.

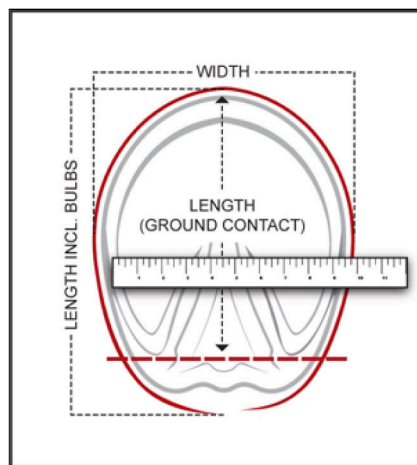

- 4** Consult the size chart to see which gel orthotic / boot size to order.

Note: Some "underslung hoofs" may need next size up

SIZE CHART

WIDTH	LENGTH	LENGTH INC. BULBS	SIZE	PART #
7 in. 178 mm	6 3/4 in. 169 mm	8 1/2 in. 217 mm		SR1001
6 5/8 in. 170 mm	7 1/4 in. 182 mm	8 in. 203 mm	2	SR1002
6 1/4 in. 155 mm	6 1/2 in. 166 mm	7 1/4 in. 185 mm	3	SR1003
5 3/4 in. 147 mm	6 in. 153 mm	6 3/4 in. 172 mm	4	SR1004
5 3/4 in. 147 mm	5 3/4 in. 147 mm	6 1/2 in. 165 mm	5	SR1005
5 1/4 in. 134 mm	5 1/4 in. 134 mm	6 in. 153 mm	6	SR1006
4 3/4 in. 121 mm	4 3/4 in. 121 mm	5 1/2 in. 140 mm	7	SR1007
4 1/2 in. 115 mm	4 1/4 in. 108 mm	4 3/4 in. 122 mm	8	SR1008
3 7/8 in. 99 mm	4 1/4 in. 108 mm	4 3/4 in. 122 mm	8	SR1008
3 5/8 in. 93 mm	3 3/4 in. 96 mm	4 1/2 in. 115 mm	9	SR1009
2 3/4 in. 70 mm	2 3/4 in. 70 mm	3 1/4 in. 83 mm	10	SR1010

**Yellow highlighted products indicate experimental products that are only available in special circumstances. Please call to order these items.

Click here to download these instructions, including the chart in a handy, printable 8 1/2"x11" PDF format. Take it with you to the stall!